

**OptiSmooth MD Hydrophilic Meatal Dilator with Tapered Tip for single use only**



Thank you for choosing the OptiSmooth hydrophilic coated Meatal Dilator. It has been especially developed from many years of listening to people and by consulting experienced healthcare professionals, we think it is one of the most advanced Meatal Dilators available today.

This product is made of medical grade, high quality thermosensitive PVC and is intended for single use only. The OptiSmooth Meatal Dilator incorporates a very smooth lubricious hydrophilic coating developed for optimum smoothness. We feel this could potentially improve the comfort of insertion and removal.

**Product codes**


14cm 08ch	OSMDIL08	14cm 14ch	OSMDIL14
14cm 10ch	OSMDIL10	14cm 16ch	OSMDIL16
14cm 12ch	OSMDIL12	14cm 18ch	OSMDIL18

**Intended Use**

This product is to be used for Dilatation purposes only.

**Precautions**

- This product should be used solely for Dilatation only , upon the recommendation of a qualified healthcare professional.
- For single use only. Do not reuse.
- Do not use if the package or product has accidentally been damaged or contaminated.
- Use immediately after opening. After using please dispose of the catheter by placing the used catheter back into the opened packaging and dispose the pack safely and responsibly.

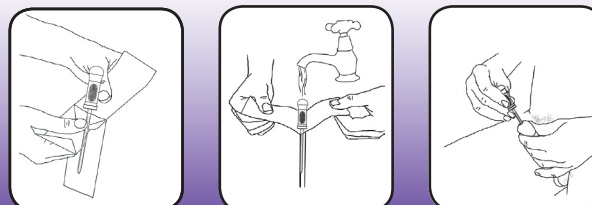

**Recommended method of use**

1. Begin by washing your hands and the urethral opening or Stoma (Opening) to help prevent bacterial contamination.
2. Open the packet approximately 2-4 cm and add sufficient water to the pack (sterile water or saline can also be used) . This will ensure your Dilator is fully immersed in water and the coating has been fully activated. Please leave the Dilator for 15-30 seconds for the hydrophilic coating to become activated.  
USE THE STICKY PAD TO SECURE THE DILATOR PACK TO A SURFACE WHILST YOU PREPARE FOR THE PROCEDURE.
3. When ready peel back the packaging further and remove the Dilator. Please take care not drop or contaminate the Dilator.
4. Position the OptiSmooth Meatal Dilator at the urethral or Stoma Opening . Use your thumb and two fore fingers to grip the insertion aid.
5. Slowly insert the tip of the Dilator down into the urethra. You will be advised how far down by your health care professional
6. After you have successfully passed the urethral tight spot, slowly remove the Dilator and place in the empty packet and dispose responsibly into a waste bin.

**Storage recommendation**

Do not store at extreme temperatures and avoid humidity. Avoid direct sunlight.

# OptiSmooth MD®

Optimum smoothness Optimum comfort

## OptiSmooth MD®

Optimum smoothness Optimum comfort

OptiSmooth MD Hydrophilic Meatal Dilator  
with Tapered Tip for single use only

**Hangzhou Bever Medical Devices Co., Ltd.**  
Add: Building 2, No. 1-1, Houmuqiao, Yongle Village,  
Cangqian Street, Yuhang District, Hangzhou,  
China. 311121

**EC REP Shanghai International Holding Corp. GmbH (Europe)**  
Add: Eifflstrasse 80D-20537 Hamburg, Germany

# Vesica Urology Ltd

# OptiSmooth MD®

Optimum smoothness Optimum comfort

## INSTRUCTIONS FOR USE

### Vesica Urology Ltd

82 High Street  
Sandhurst  
Berkshire  
GU47 8EE  
Phone: +44 (0) 1753 445087  
Fax: +44 (0) 1753 447446  
adrian@vesicaurology.com  
www.vesicaurology.com

<b>APPROVED ARTWORK/批准版面</b>	
Controlled Doc. /受控文件	HMD-171IFU 001 Rev: B 23/01 超滑尿道扩张器说明书客户171版面
Dimension (mm) /尺寸(毫米)	450 × 95 ( 正反两面 )
Color/颜色	P2597C/Black ( 黑色 )
Validation /有效期	3 Years 三年
Material /材质	157g铜版纸
Approval Ref. /批准索引	Mail dated on Jan. XX, 2023 2023年1月XX日邮件