

**OptiSmooth Mitro Catheter with 4 XL (xtra large)  
heat polished smooth drainage eyes for Single Use Only**



Thank you for choosing the OptiSmooth Mitro catheter with 4 XL drainage eyes. It has been specially developed from many years of listening to people who catheterise. Consulting experienced healthcare professionals we think it is one of the most advanced intermittent catheters available today.

This product is made of medical grade, high quality thermosensitive PVC and is intended for Single Use Only.

The OptiSmooth Mitro catheter incorporates 4 XL heat polished staggered drainage eyes developed to improve urine drainage.

Product type or specification

40cm OSMXL4010 OSMXL4012 OSMXL4014

**Intended Use**

This product is to be used for intermittent self-catherisation (ISC) purposes and urine drainage purposes only

**Precautions**

- This product should be used solely for intermittent catheterisation , upon the recommendation of a qualified healthcare professional
- For single use only. Do not reuse.
- Do not use if the package or product has accidentally been damaged or contaminated.
- Use immediately after opening. After using please dispose of the catheter by placing the used catheter back into the opened packaging and dispose the pack safely and responsibly

**Recommended method of use**

1. Begin by washing your hands
2. Open the packet and take out the catheter using the funnel. Coat the catheter with a water soluble gel lubrication.
3. Position the OptiSmooth Mitro catheter at the urethral or Stoma (Opening/channel) with one hand; using the other hand to insert the lubricated catheter gently down into the urethra or passage until urine starts to flow.  
  
You can drain the urine straight into the toilet or a receptacle or you can connect the OptiSmooth funnel to a urine drainage bag if preferred.
4. When the bladder is completely empty, slowly remove the catheter and dispose of it by placing it in the empty product pack. Please dispose of the product in an appropriate waste bin

Storage recommendation  
Do not store at extreme temperatures and avoid humidity.  
Avoid direct sunlight

**OptiSmooth<sup>®</sup> Mitro<sup>®</sup> XL<sup>®</sup>**

*Optimum smoothness Optimum comfort*

OptiSmooth<sup>®</sup> is a registered trademark, Mitro<sup>®</sup> is a registered trademark  
OptiSmooth Mitro is a UK unregistered Design 2011



OptiSmooth Mitro Catheter with 4 XL (xtra large) drainage eyes  
for Single Use Only

**Hangzhou Bever Medical Devices Co., Ltd.**  
Add: Building 2, No. 1-1, Houmuqiao, Yongle Village,  
Cangqian Street, Yuhang District, Hangzhou,  
China. 311121

**EC REP Shanghai International Holding Corp. GmbH (Europe)**  
Add: Eiffestrasse 80D-20537 Hamburg, Germany



**OptiSmooth<sup>®</sup> Mitro<sup>®</sup> XL<sup>®</sup>**  
*Optimum smoothness Optimum comfort*

**Vesica Urology Ltd**

**OptiSmooth<sup>®</sup> Mitro<sup>®</sup> XL<sup>®</sup>**  
*Optimum smoothness Optimum comfort*

**INSTRUCTIONS FOR USE**

**Vesica Urology Ltd**

PO Box 4145  
Slough  
Berkshire  
SL1 0QA

Phone: +44 (0) 1753 445087  
Fax: +44 (0) 1753 447446  
adrian@vesicaurology.com  
www.vesicaurology.com

<b>APPROVED ARTWORK/批准版面</b>	
Controlled Doc. /受控文件	NC-1711FU 002 Rev: A 18/03 普通导尿管 (超大四孔) 说明书客户171版面
Dimension (mm) /尺寸 (毫米)	450 × 95 (正反两面)
Color/颜色	P2597C/Black (黑色)
Validation /有效期	4 Years 四年
Material /材质	157g铜版纸
Approval Ref. /批准索引	Mail dated on Mar. xx, 2018 2018年3月xx日邮件